**DEWIMED**<sup>®</sup>  
SMART INNOVATION

**O.S.A.S.**

**Orthodontic Skeletal Anchorage System**

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**O.S.A.S.** Orthodontic Skeletal Anchorage System

## MORE EFFECTIVE ORTHODONTICS

The future of orthodontic anchorage implants is the mini TITANIUM screw. In contrast to traditional treatment there is no need for anchorage teeth anymore, so teeth movement can be realized even if there are teeth missing.

Thanks to DEWIMED O.S.A.S. MINI Screw extra oral equipment becomes unnecessary. They allow effective working without being dependent on cooperation of individual patients. Minimal size, as well as the complication-free use of mini screws opens an innovative clinical therapy spectrum. This means biomechanically creative solutions, a shorter treatment period and effective tooth movements become a common way of working for orthodontics.

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## ALL-ROUND SYSTEM

Every DEWIMED O.S.A.S. MINI Screw is made of TITANIUM (Ti6Al4V) which is the globally successful, safe and experienced material with particularly high biocompatibility.

The direct implant fixation allows an immediate transmission of forces to the implant. In general, the implants are loaded immediately after insertion. In contrast to conventional implants DEWIMED O.S.A.S. MINI Screw are **not** completely osseointegrated. Temporary limited application time for orthodontic purposes with its incomplete integration is not clinically detrimental - it makes the later implant removal even easier.

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**Dr. Ramón Mompell**  
España

### ... optimal insertion

The German quality of DEWIMED O.S.A.S. MINI Screw is evident from the moment they are cast. Their self-drilling tip allows an optimal insertion, guaranteeing their primary stability by not damaging the tissues during the collation. Its different models and heads also cover all the need that can be raised in a dental clinic for the resolution of complex malocclusions without the need to resort to other complementary systems.

Its removal is easy and painless for the patient because the DEWIMED O.S.A.S. MINI Screw retain their properties until the end of treatment, and therefore does not tear the tissue during removal of them. The screwdriver is easy to use and complement the good experience for both the professional and the patient.

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**...clinical stability**

After having used since 2009 different types of DEWIMED O.S.A.S. MINI Screws and resolved various types of malocclusions with them, I can recommend them widely for clinical stability, tolerance by patients and easy insertion.

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**Prof. Dr. Ricardo Medellín**  
México

**...shorter treatment duration**

The DEWIMED O.S.A.S. MINI Screw is a great product! The easy insertion and the flat head make it useful for placement in several areas of the jaws. I have been using the O.S.A.S. system since 2007. Excellent quality in cutting the active tip and thread. Many students tell me about the easy insertion.

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**Dr. Gustavo Gregoret**  
Argentina

**...simple and best**

I am one of the first users of DEWIMED O.S.A.S. MINI Screws in Latin America (since 2006), and I have compared the clinical performance of different micro screws from several research laboratories. I can affirm that thanks to the precision of the body of the mini screw we achieved a very simple insertion technique. This allows orthodontists, even without any previous experience, to use them to solve highly complex clinical cases.

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**Prof. Dr. Jose Antonio Villavicencio Limón**  
México

**...when you know the best you only want the best**

It is a constant between researchers, academics and clinicians that when you know the best you only want the best. And if in addition these values of excellence are supported by the investigation and quality control of first level, the DEWIMED O.S.A.S. MINI Screw are an indispensable tool in the Biomechanics of Innovation in benefit of the patient.

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### ...clinical management is simple

Working with DEWIMED O.S.A.S. MINI Screws since 2008 has allowed me to significantly improve my orthodontic practice through creating simple, efficient and predictable clinical protocols of highly complex dental movements. These protocols include intrusion, retraction, open bite closure, etc. and are characterized by the manipulation of the line of action of the resultant force in relation to the center of resistance of the active unit in order to create statically determined force systems. Manipulating forces in relation to the center of resistance is the key for clinical success.

Clinical handling of DEWIMED O.S.A.S. MINI Screws is simple. There is a short learning curve to achieve mastery in insertion. After inserting more than 700 DEWIMED O.S.A.S. MINI Screws on my patients, I can assure that fracture is virtually reduced to zero when manipulation is precise. After all, I have not fractured any mini-implant yet.

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Perú

# SMART MADE INNOVATION IN GERMANY

## **MADE IN GERMANY** HECHOS EN ALEMANIA MADE IN GERMANY

DEWIMED O.S.A.S. MINI Screws are made of one piece TITANIUM alloy and are more stable mechanically due to their homogeneous material structure. The joint-free one-piece technology guarantees particular mechanical stability and protects against material defects. Screw bodies composed of joined parts involve a risk of fracture on incorrect loading in the form of a break point. O.S.A.S. System is produced 100% in Germany according to German precise technology.

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## PERFECT SOLUTION SOLUCIÓN PERFECTA PERFEKTE LÖSUNG

Each orthodontic treatment is individual and unique, therefore the mini screws must be individual and adapted to each type of treatment. We have 8 different types of heads. The bone structure, the insertion area and the condition of the gums are decisive factors for screw selection.

The diameter of the screw is usually 1.6 mm. The O.S.A.S. Mini is available in 1.3 mm, for optimal treatment in the anterior mandible and the O.S.A.S. 2.1 with a diameter of 2.1 mm, for optimal treatment in the palatal region.

All DEWIMED O.S.A.S. MINI Screws have a self-drilling and self-tapping tip for easy and safe insertion into the bone.

All DEWIMED O.S.A.S. MINI Screws are compatible with the complete screw program so using the screw driver and one tip enables to work with the complete range of screws.

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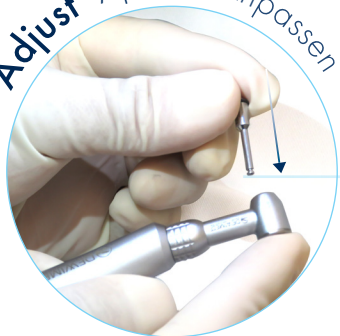
Accesorios

Zubehör

### Ratchet and accessories for optimal adaption for the lingual position of the screw

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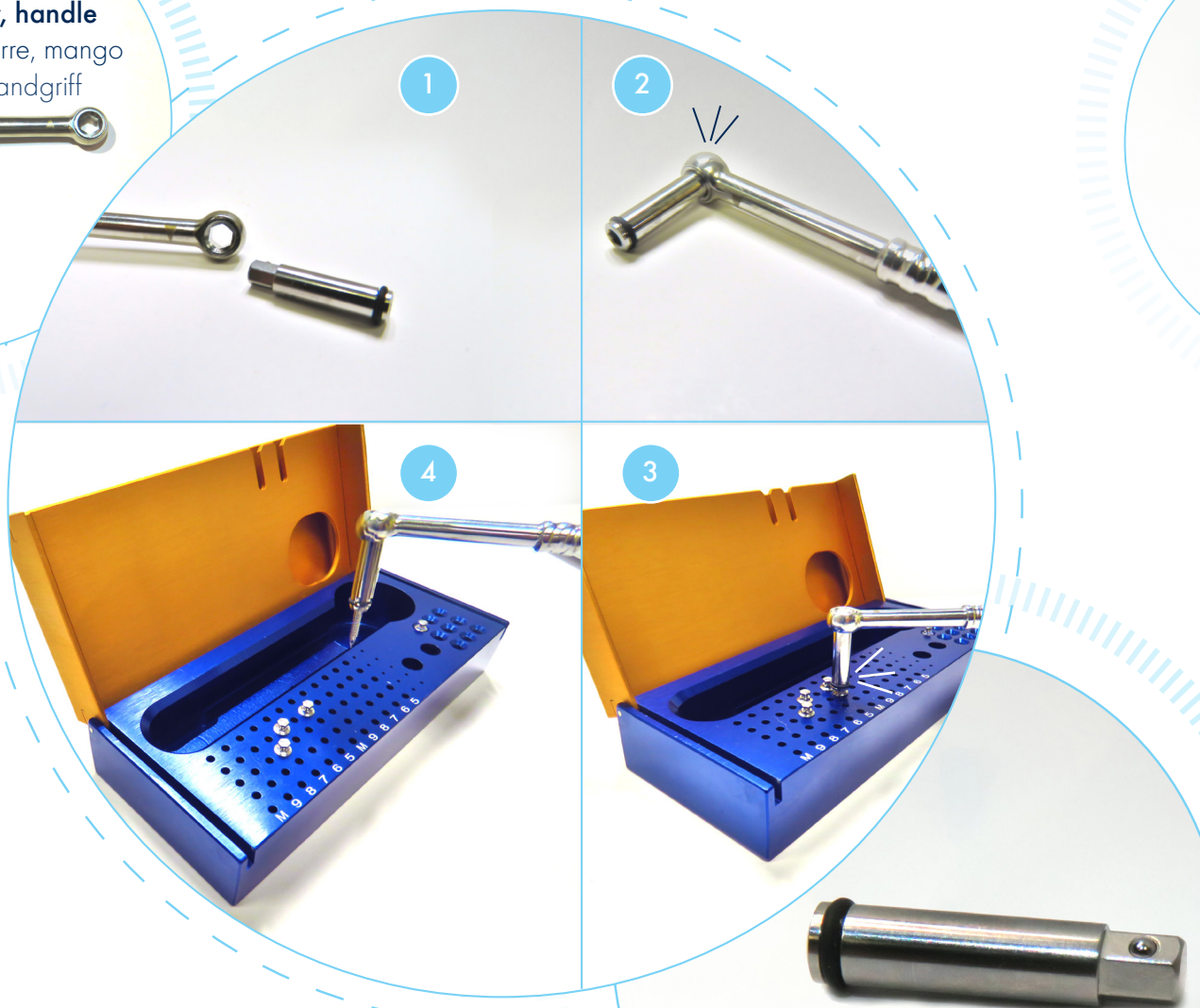
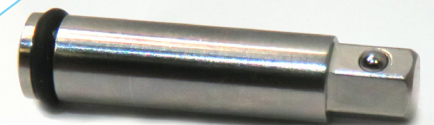
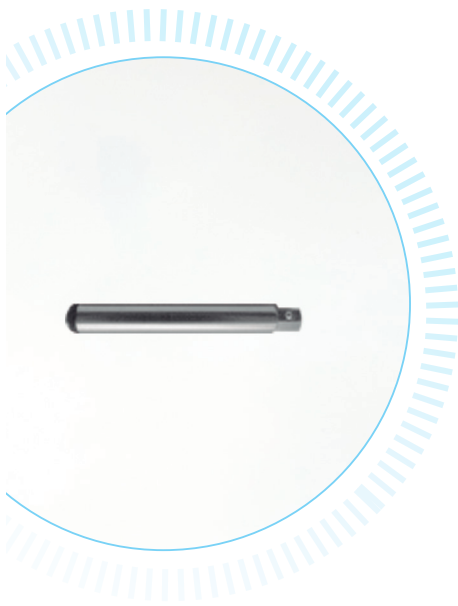
Klinge für Ratschenschlüssel, 30 mm

**10-0100**

### Repair Kit for for Screwdriver tip 40 balls, 20 O-rings

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## Racks

Racks  
Racks

### Storage of DEWIMED O.S.A.S. MINI Screws and instruments. Additional support to check the length of DEWIMED O.S.A.S. MINI Screws.

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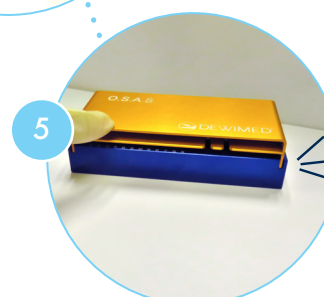
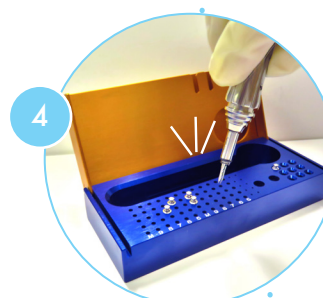
**Manually turned handpiece no / better use or**

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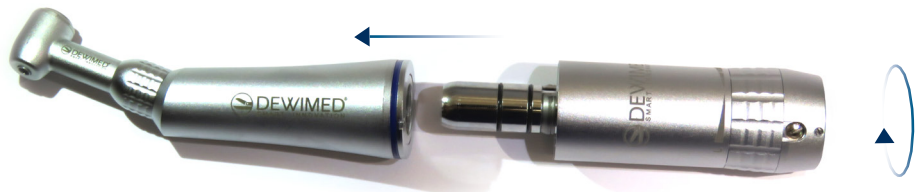
**Handpiece of the manual adjustment model Pauls, With adjustable torque from 0 to 50 Ncm.**

Con torque ajustable desde 0 a 50 Ncm. Pieza de Mano de ajuste manual modelo a Pauls. Handstück des manuellen Verstellmodells Pauls, mit einstellbarem Drehmoment. von 0 bis 50 Ncm.

The manually turned handpiece is connected to your existing angled motor handpiece (blue 1:1) which allows it to be used like an angled screwdriver.

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**10-0018**

Manually turned handpiece mod. to Pauls with adjustable torque from 0 - 50 Ncm.

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**10-0017**

The manually turned handpiece for angled screwdriver with lightsource.

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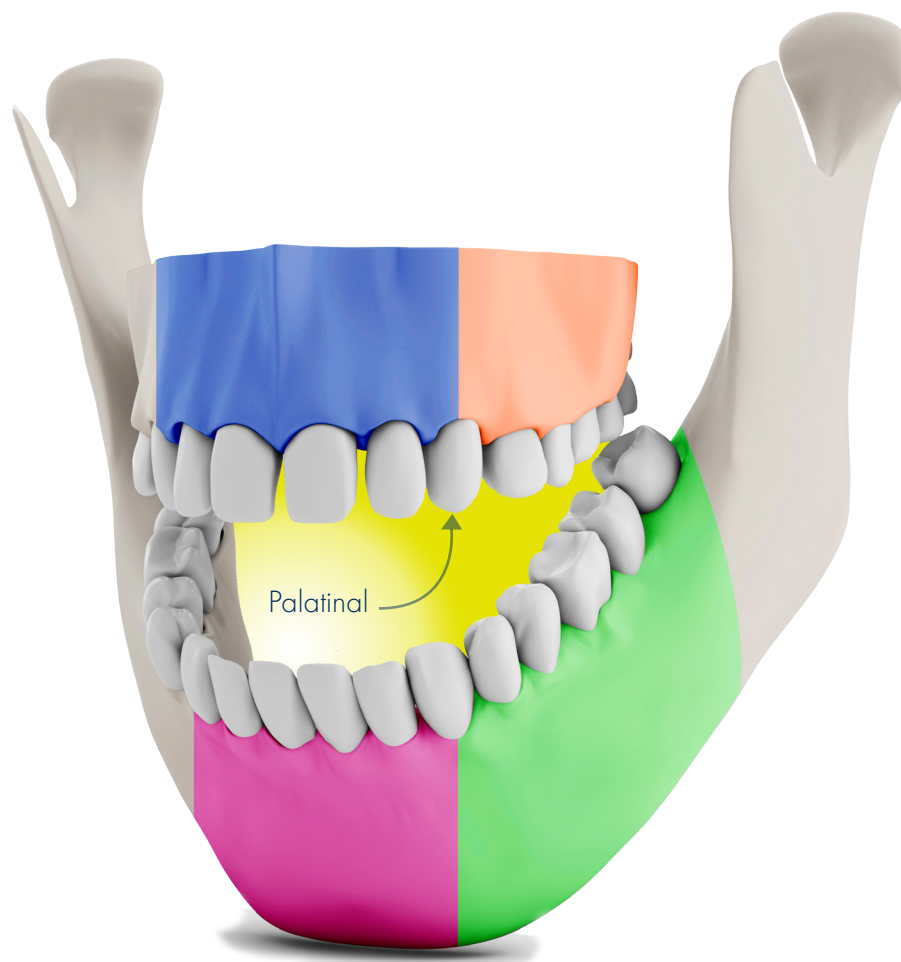
## Advantages of the DEWIMED O.S.A.S. System

Ventajas del sistema DEWIMED O.S.A.S.

Vorteile des DEWIMED O.S.A.S. Systems

Self-drilling screw systems, such as the DEWIMED O.S.A.S. System, do not require any preparation of the bone. The thread works through the gingiva and cortex without pre-drilling because of its self tapping tip. Punching the gingiva to prepare a passage is not necessary either with the O.S.A.S. System. In most situations, infiltration anaesthesia can be omitted and the screw can be inserted with surface anaesthesia.

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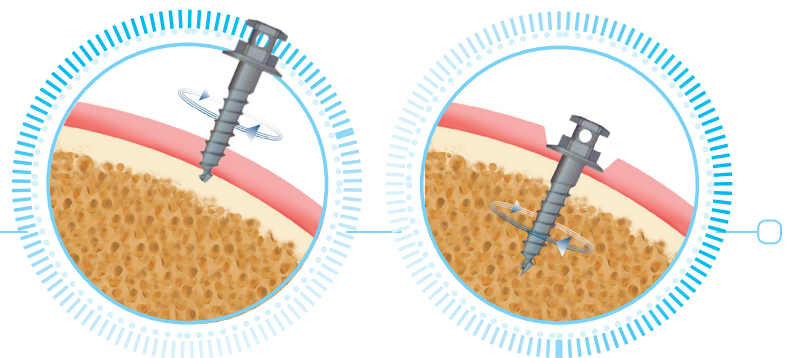
### O.S.A.S. System self cutting, self - drilling and easy

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
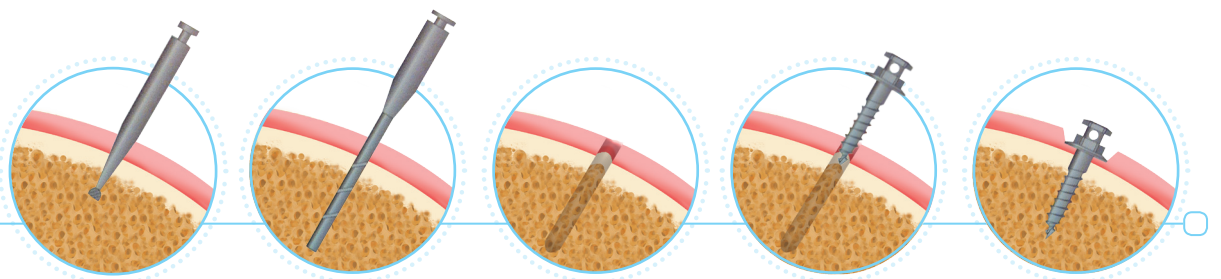
The DEWIMED O.S.A.S. MINI Screw may be loaded functionally without a healing period. Immediate loading has a beneficial effect on contact with the surrounding bone and promotes the german quality of the anchorage. Self-drilling and tapping to facilitate placement, the shape of the head inhibits the overgrowth of the gingival. With only a few DEWIMED O.S.A.S. MINI Screw, strategically placed it will have creative solutions of orthodontic and orthopaedic.

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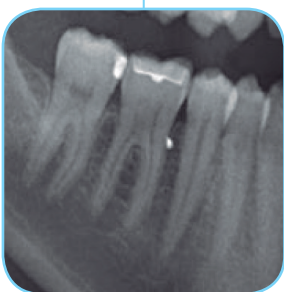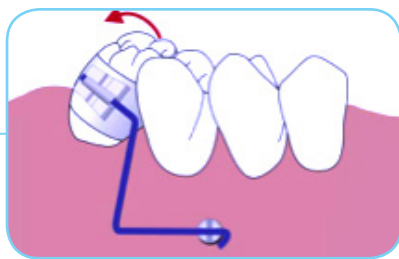
In principle, a direct or indirect strategy can be chosen. The direct variant is often easier technically and tooth movements are carried out directly supported by the stationary anchorage. Indirect anchorage means that a dental anchorage segment is formed that is stabilised through the screw, so that active tooth movements take place.

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

**Direct anchorage**  
Anclaje directo  
Direkte Verankerung

**Indirect anchorage: The tooth to be moved is anchored to a tooth that is connected to the DEWIMED O.S.A.S. MINI Screw.**

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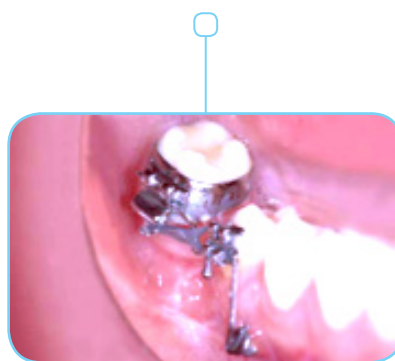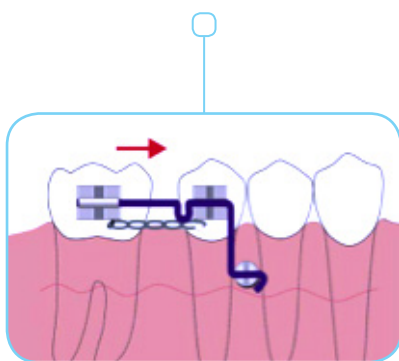## Clinical Cases

Casos Clínicos

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A 18 year old, male patient has protrusion of the upper and lower lips. With help of DEWIMED SLOT Screw on the labial sides of the maxilla and the mandible and an mini-implantation on the palatal side of the maxilla the tooth could be moved in the wished position. The orthodontic treatment was done within 23 months.

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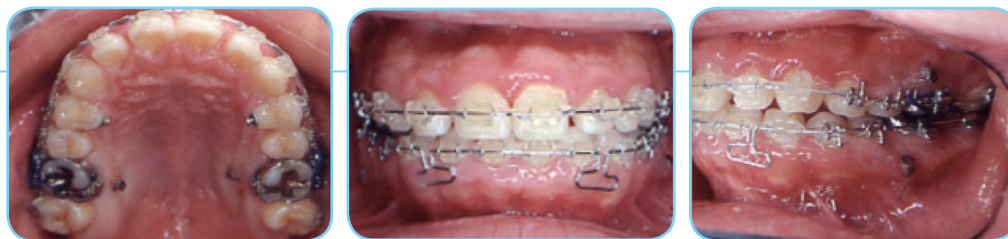
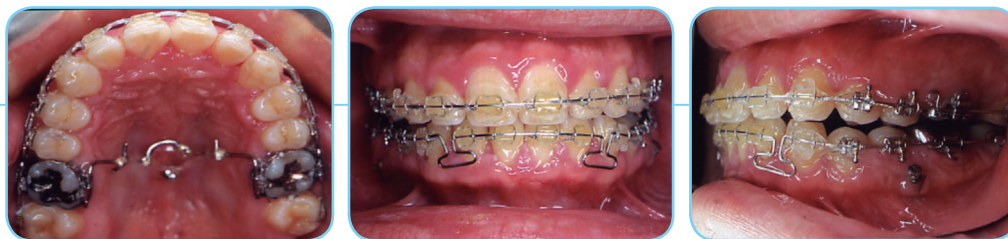
Class II 26 year old male patient (Fig. 1). Unfortunately at the beginning of its orthodontic treatment the extraction of mandibular right first molar was required by his condition (Fig. 2).

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At a patient having this bone characteristics is not easy to mesialize the second molar and take the third as option for restore posterior occlusion with traditional orthodontics. So it was necessary to place a DEWIMED O.S.A.S. MINI Screw (DEWIMED M.O.S.A.S. 8 mm) to support the mesialization additionally to its elastic class II (Fig. 3). After 11 months of treatment the space was closed and the sagittal relationship preserved as the presence of the third molar which now serves as second molar (Fig. 4).

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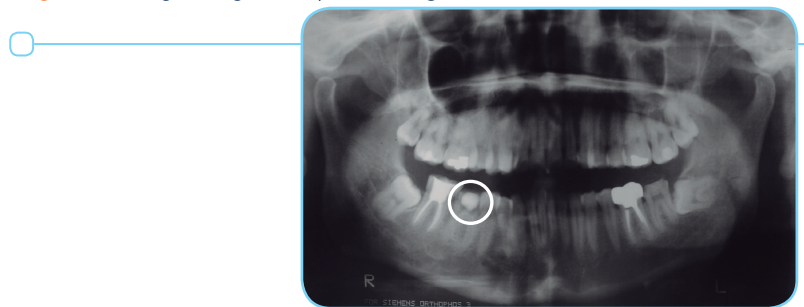
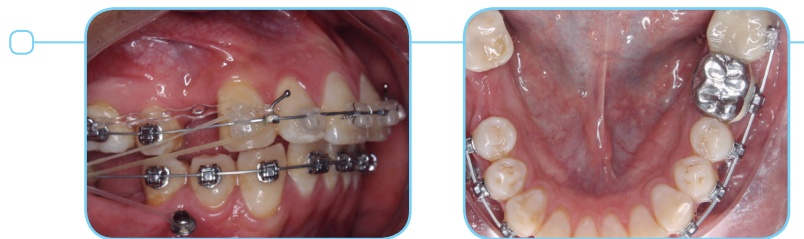
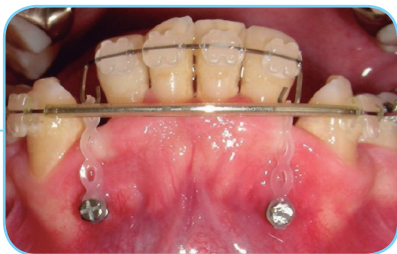
A 31 year old male patient needed lower incisor intrusion. Two mini-implants (DEWIMED 6 mm) were inserted distal to both lateral incisors to create a statically determined force system. 40g of force was vertically applied to each lateral incisor. In order to avoid vestibular inclination during intrusion a 0,018 x 0,025" stainless steel full arch wire was used. Periodontal health and teeth vitality were not compromised during intrusion. The complete treatment was finished after 4 months.

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A 23 year old female patient with multiple missing posterior teeth needed anterior teeth retraction. 4 DEWIMED O.S.A.S. MINI Screw (DEWIMED, M.O.S.A.S. 6 mm and SLOT 6 mm) were vertically inserted on the edentulous alveolar ridge. Mini-implants heads were joined each other using composite resin to create "dental crowns" which let us bond an adhesive 0,022 x 0,025" tube. Crowns worked as bone anchorage for retraction mechanics.

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## Clinical Cases

Casos Clínicos

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A 11 year old female patient needed unilateral Brodie bite correction. Intrusion and palatal inclination of the upper left first molar and second bicuspid was planned to harmonize the transverse discrepancy. Two mini-implants DEWIMED, M.O.S.A.S. 6 mm were inserted 2 mm away from the midpalatal raphe between first and second bicuspids. The heads of the mini-implants were joined to an acrylic button to create a bone anchorage unit. Two hooks were strategically placed into the acrylic and two lingual buttons were bonded on the palatal surfaces of the first molar and second bicuspid. 50g of force was individually applied to each tooth. Full fixed appliances were used to correct lower first molar lingual inclination and malocclusion.

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All of the familiar coupling elements used in orthodontics (round wires, rectangular wires, springs and elastic chains) can be used. Each orthodontist can maintain his treatment strategy and use the advantages of cortical anchorage. Removal after successful treatment is unproblematic.

As the fixation of the **non**-osseointegrating DEWIMED O.S.A.S. MINI Screws is reduced after a few months as intended, the screws can be removed easily with a screwdriver, usually without anaesthetic. The screwdriver fixes the screw by locking so that the screw cannot slip off into the pharynx. As the dimensions of the screw are a mechanical minimum, they leave no trace after removal and guarantee rapid mucosa-covered healing of the intraoral penetration site.

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### Healing period of 4 days after removal

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**DEWIMED Medizintechnik GmbH**