CE₀₁₉₇ The EU directive 93/42/EEC was applied in the design and production of this medical device.

NAKANISHI INC. 

www.nsk-inc.com

700 Shimohinata Kanuma-shi
Tochigi 322-8666,
Japan

NSK America Corp

www.nskdental.com

1800 Global Parkway
Hoffman Estates, IL 60192,
USA

NSK Oceania Pty Ltd

www.nskoceania.com.au

Unit 22, 198-222 Young St.
Waterloo, Sydney,
NSW 2017, Australia

NSK Europe GmbH 

www.nsk-europe.de

Elly-Beinhorn-Strasse 8
65760 Eschborn,
Germany

NSK France SAS

www.nsk.fr

32 rue de Lisbonne
75008 Paris,
France

NSK Middle East

www.nsk-inc.com

Room 6EA-701, 7th Floor, East Wing No.6
Dubai Airport Free Zone,
PO Box 54316 Dubai, UAE

NSK United Kingdom Ltd

www.nsk-uk.com

Office 5, Gateway1000,
Arlington Business Park, Whittle Way,
Stevenage, SG1 2FP, UK

NSK Dental Spain SA

www.nsk-spain.es

C/ Mòdena, 43 El Soho-Európolis
28232 Las Rozas, Madrid,
Spain

NSK Asia Pte Ltd

www.nsk-inc.com

1 Maritime Square,
#09-33 HarbourFront Centre,
Singapore 099253

Specifications are subject to change without notice.

2013.05.20 40 

OPERATION MANUAL

NSK

CE₀₁₉₇

Maintenance Guide


for Air Turbine, Contra Angle
& Straight Handpiece

OM-Z0135E

Please read this Operation Manual carefully
before use, and file for future reference.

Powerful Partners®

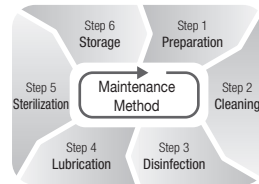This symbol on your NSK product means that the product can be cleaned and disinfected with a Thermo-disinfector.

This symbol on your NSK product means that the product can be autoclavable up to Max. 135°C.

1. Preparation

Remove Contra Angle/Turbine from the motor/coupling and detach the bur from the chuck.

2. Cleaning

Manual : Clean the external surface of handpiece with running water (<38°C, demineralised water recommended).

Automated : NSK handpieces with this symbol are the thermo-disinfector-ready and may be washed and disinfected by using a thermo-disinfector. When using a Thermo-disinfector, be sure to follow its manufacturer's instructions.

3. Disinfection

Manual : Wipe the external surface of handpiece carefully with cleaning or disinfectant solution.

Automated:

4. Lubrication


Lubricate handpiece by using NSK PANA SPRAY Plus / PANA SPRAY. To prevent mist spray from escaping into the environment, use absorbent cloth. Expel excess lubrication.

Manual :

- 1) Push or screw nozzle. Shake the spray can 3-4 times.
- 2) Insert the spray nozzle in the rear of the handpiece and spray for approximately 2-3 seconds until the oil comes out of the handpiece head / nose.

Automated :

Clean and lubricate handpieces with NSK Care3 Plus. Be sure to use a right adaptor for each handpiece. Refer to the Care3 Plus Operation Manual for detail.

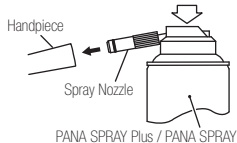CAUTION

- Be sure to hold the handpiece firmly to prevent the handpiece from slipping out by the spray pressure when lubricating.
- Keep the NSK PANA SPRAY Plus / PANA SPRAY can upright.
- Use of lubricants other than NSK's PANA SPRAY Plus / PANA SPRAY can cause excessive heat generation and possible handpiece damage.

5. Sterilization

Autoclave sterilization is recommended.

Sterilization is required first time you use and after each patient as noted below.

■ Autoclave Procedure :

- 1) Insert into an autoclave pouch. Seal the pouch.
- 2) Autoclavable up to a max. 135°C.
ex.) Autoclave for 20 min. at 121°C, or 15 min. at 132°C.

*Sterilization at 121°C for more than 15 minutes is recommended by EN13060 or EN ISO17665-1.

CAUTION of Sterilization

- Always place the handpiece in the center or upper shelf of the chamber, as the local temperature at the bottom of chamber could rise beyond the set value.
- Do not autoclave the handpiece even when it is bagged, together with other instruments, from which chemicals are not sufficiently cleaned. This is to avoid discoloration and/or damage the plating of the handpiece/motor.
- Do not heat or cool the handpiece quickly. Rapid change in temperature could break the glass rod and give abnormal strain to other metals. (Optic handpiece/motor)
- The handpiece is kept in suitable atmospheric pressure, temperature, humidity, ventilation, and sunlight. Moreover, air does not include a dust, salt and sulfur.
- We do not recommend you to sterilize on plasma sterilization or EOG sterilization.

6. Storage

Immediately after a sterilization cycle, remove handpiece from the autoclave. Keep them in a dust-free sterilized storage or bring them to a treatment room for use.

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