



Recommended drilling sequence guide

for Interna[®] BTI implants

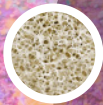

Drill depth marks

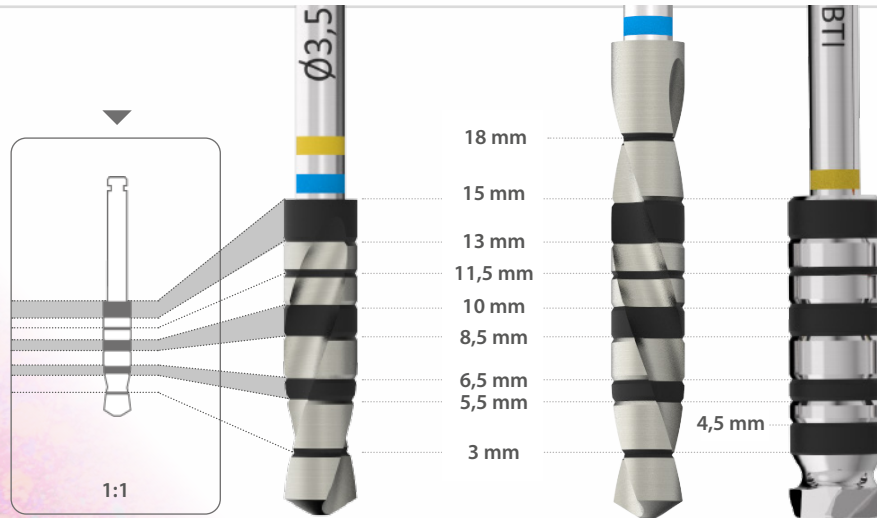

Class/Hounsfield	III	IV	V
I	1050 - 1400	400 - 500	100 - 350
II	850 - 1000	550 - 800	400 - 500

DRILL	REVOLUTIONS	IRRIGATION
Initial drill	800 - 1.000 rpm	yes
Drill	50 - 75 rpm	no
Countersink drill	150 - 200 rpm	no
Front cutting drills	50 - 75 rpm	no




Standard length implants

(from 8,5 to 13 mm)



- Total length of the implant
- 75 % of the total length of the implant
- 25 % of the total length of the implant

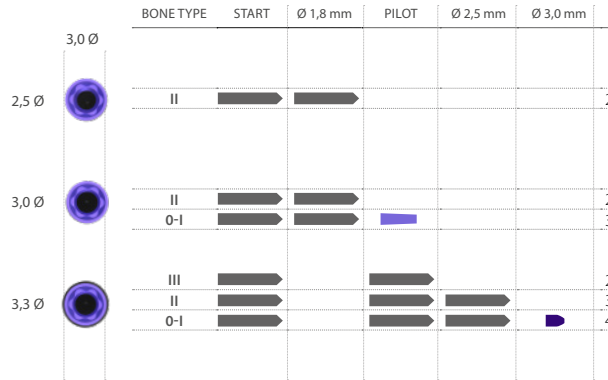

3.0 Platform
Platform diameter Ø3,0 mm



drilling
sequence guide






Short implants

(5,5 - 6,5 - 7,5 mm)

 Total length of the implant
 2-3 mm drilling depth

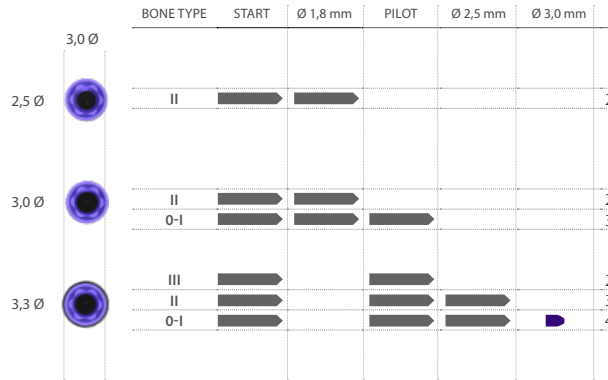





3.0 Platform
 Platform diameter Ø3,0 mm

drilling
sequence guide



Standard length implants

(from 8,5 to 15 mm)

 Total length of the implant
 6,5 mm drilling depth
 2-3 mm drilling depth







Narrow/CORE Platform
 Platform diameter Ø3,5 mm

drilling
sequence guide





Short implants (4,5 - 5,5 - 6,5 - 7,5 mm)

 Total length of the implant
 Total length of the implant with specified diameter drill
 2-3 mm drilling depth




Narrow/CORE Platform
 Platform diameter Ø3,5 mm

drilling
sequence guide






Standard length implants

(from 8,5 to 15 mm)

- Total length of the implant
- 75 % of the total length of the implant
- 25 % of the total length of the implant
- 5,0 mm drilling depth

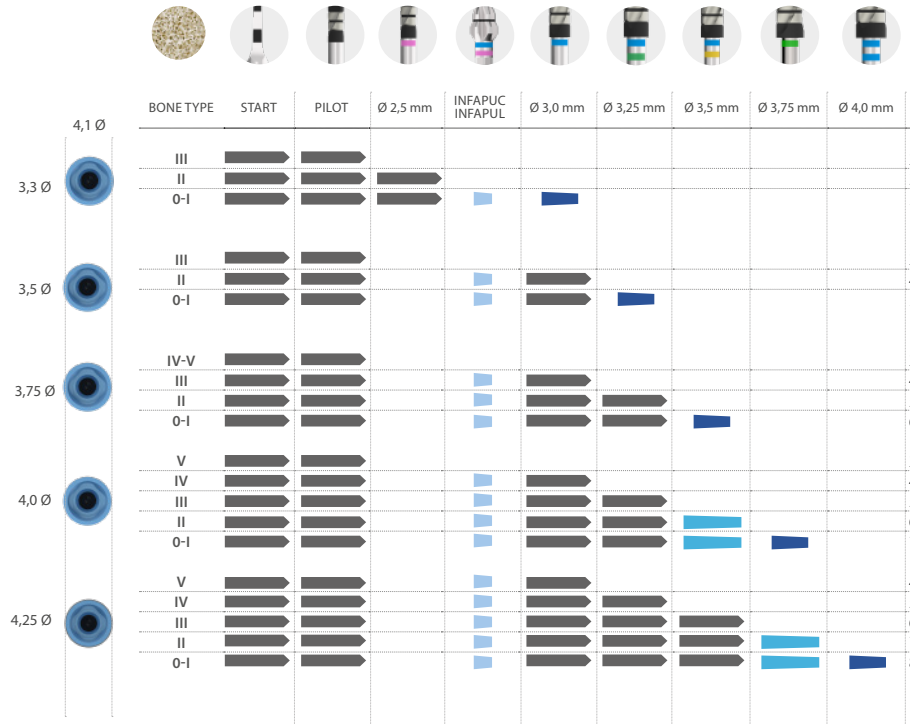

Universal Platform
Platform diameter Ø4,1 mm

drilling
sequence guide






Short implants

(5,5 - 6,5 - 7,5 mm)

- Total length of the implant
- Total length of the implant with specified diameter drill
- 5,0 mm drilling depth
- 2-3 mm drilling depth

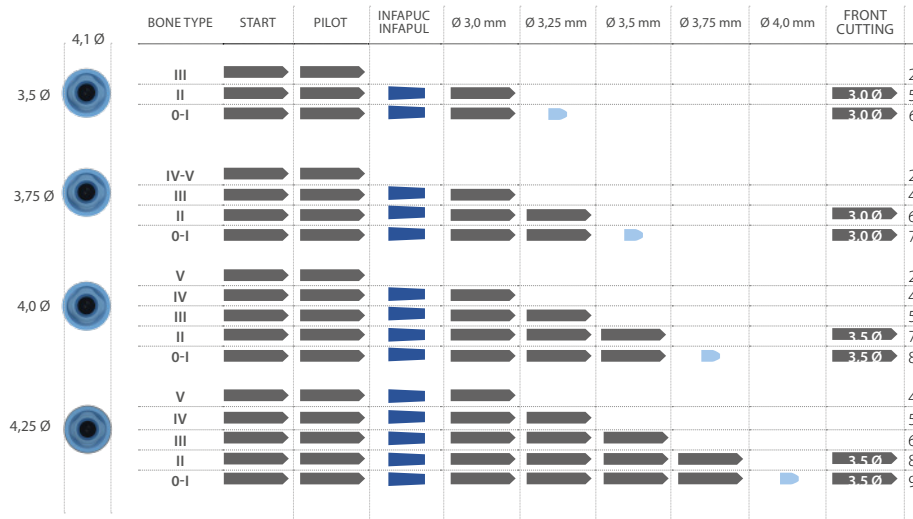

Universal Platform
Platform diameter Ø4,1 mm

drilling
sequence guide








Standard length implants

(from 8,5 to 15 mm)



 Total length of the implant
 6,5 mm drilling depth
 3 mm drilling depth







Universal Plus Platform
 Platform diameter Ø4,1 mm


drilling
sequence guide



Short implants

(4,5 - 5,5 - 6,5 - 7,5 mm)

 Total length of the implant
 Total length of the implant with specified diameter drill
 3 mm drilling depth

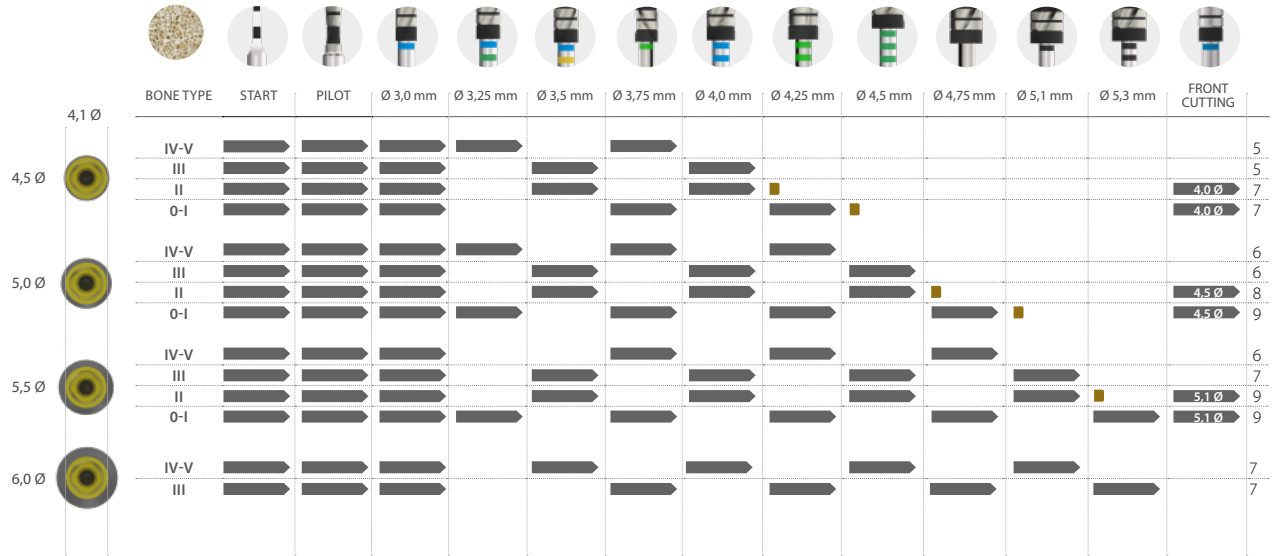






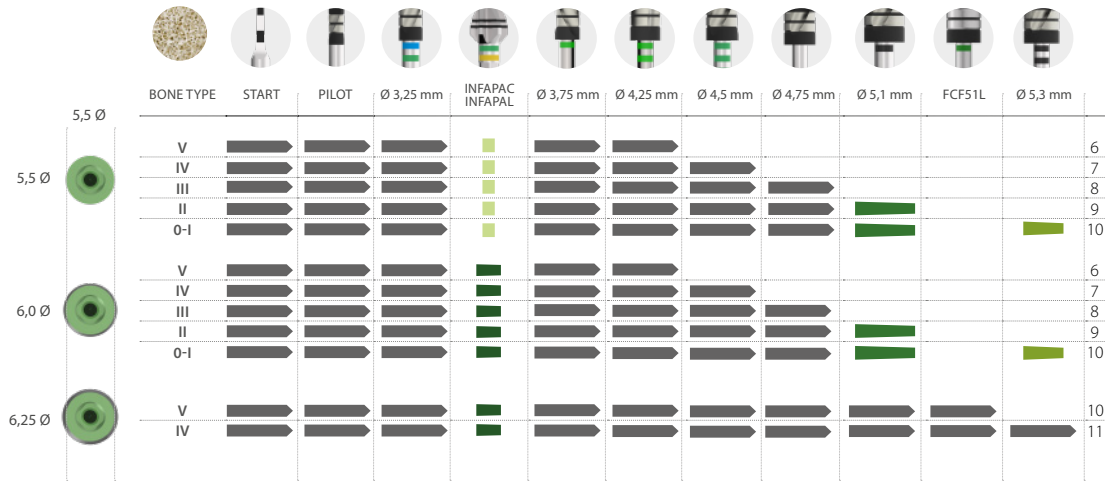


Standard length implants

(from 8,5 to 15 mm)



-  Total length of the implant
-  75 % of the total length of the implant
-  25 % of the total length of the implant
-  5,5 mm drilling depth
-  4,5 mm drilling depth



Wide Platform
Platform diameter Ø5,5 mm

drilling
sequence guide






Short implants

(5,5 - 6,5 - 7,5 mm)

- Total length of the implant
- Total length of the implant with specified diameter drill
- 5,5 mm drilling depth
- 4,5 mm drilling depth
- 2-3 mm drilling depth

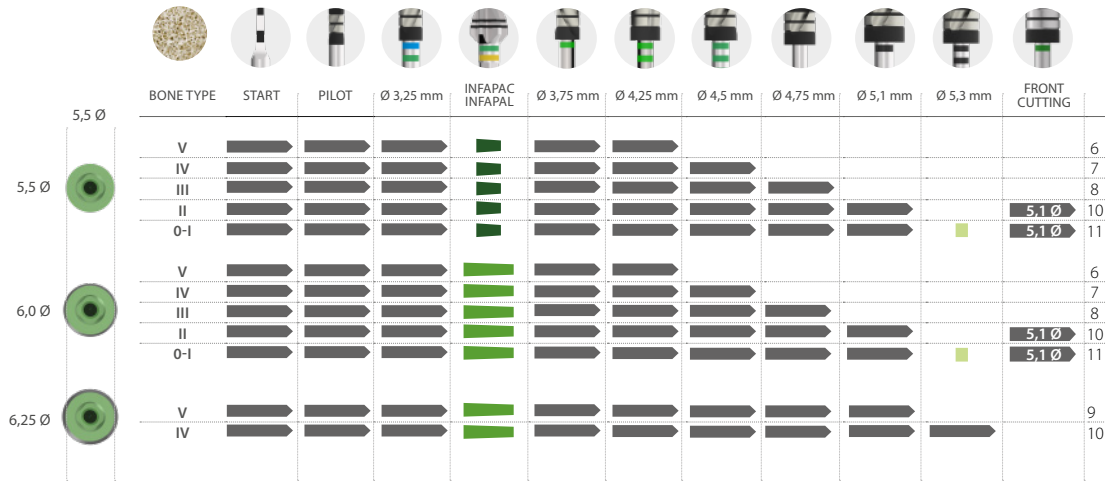

Wide Platform
Platform diameter Ø5,5 mm

drilling
sequence guide

