





Recommended drilling sequence guide

for Interna® BTI implants



Drill depth marks

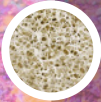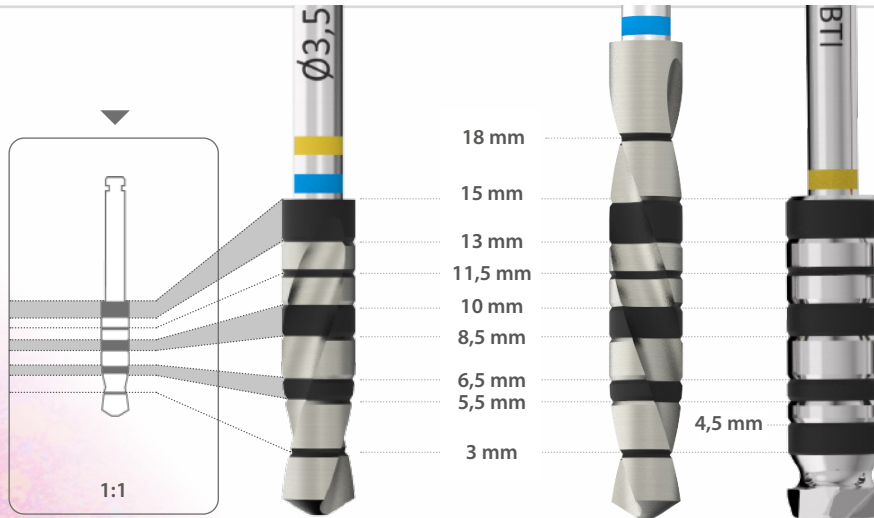

Class/Hounsfield

I 1050 - 1400

II 850 - 1000

III 550 - 800

IV 400 - 500

V 100 - 350

DRILL

Initial drill

Drill

Countersink drill

Front cutting drills

REVOLUTIONS

800 - 1.000 rpm

50 - 75 rpm

150 - 200 rpm

50 - 75 rpm

IRRIGATION

yes



no

no

no

Standard length implants

(from 8,5 to 13 mm)

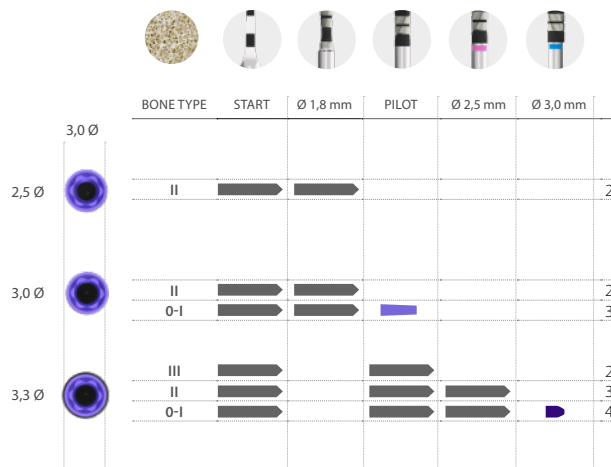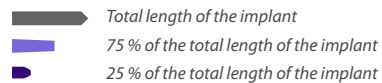


Short implants

(5,5 - 6,5 - 7,5 mm)



 Total length of the implant
 2-3 mm drilling depth

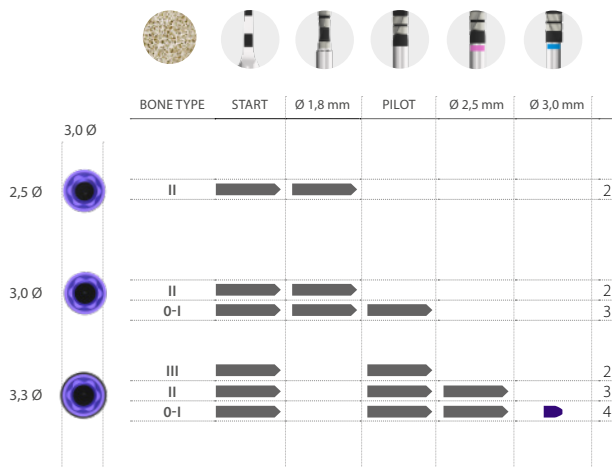




Standard length implants

(from 8,5 to 15 mm)

— Total length of the implant
— 6,5 mm drilling depth
■ 2-3 mm drilling depth

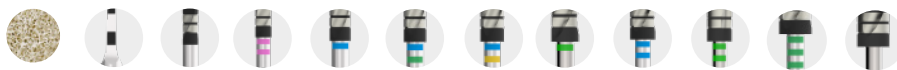


○ Narrow / CORE Platform
Platform diameter: Ø3,5 mm

drilling
sequence guide





Short implants (4,5 - 5,5 - 6,5 - 7,5 mm)

— Total length of the implant
— X.X Ø Total length of the implant with specified diameter drill
■ 2-3 mm drilling depth

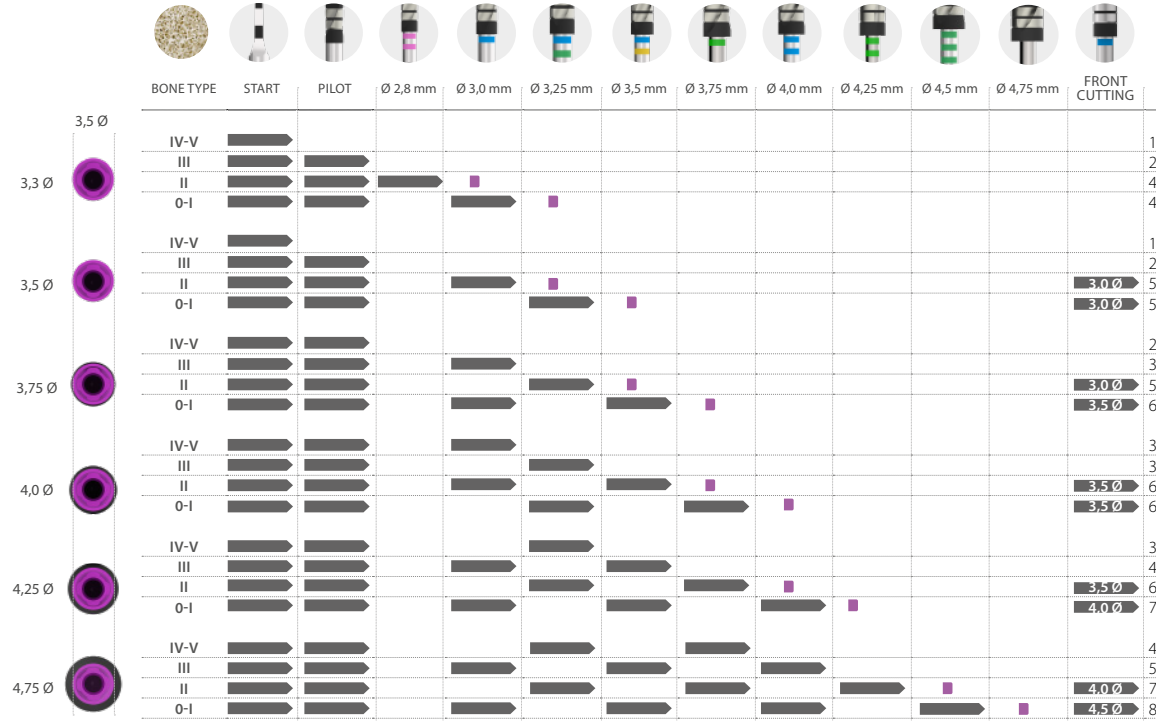

Standard length implants

(from 8,5 to 15 mm)

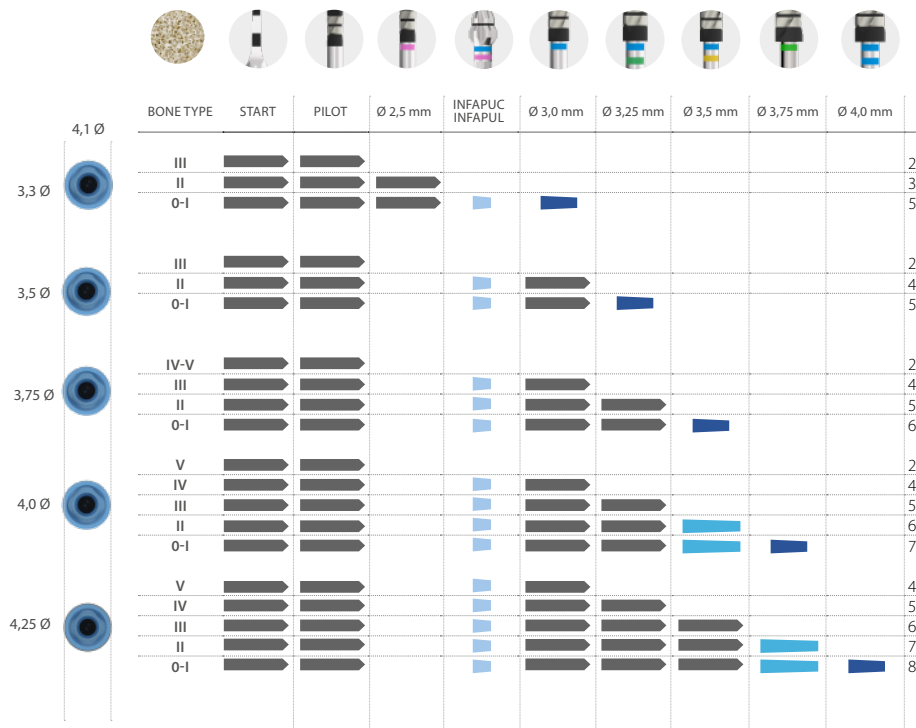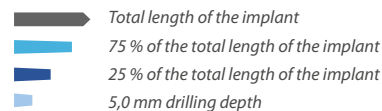




Short implants

(5,5 - 6,5 - 7,5 mm)

- Total length of the implant
- Total length of the implant with specified diameter drill
- 5,0 mm drilling depth
- 2-3 mm drilling depth

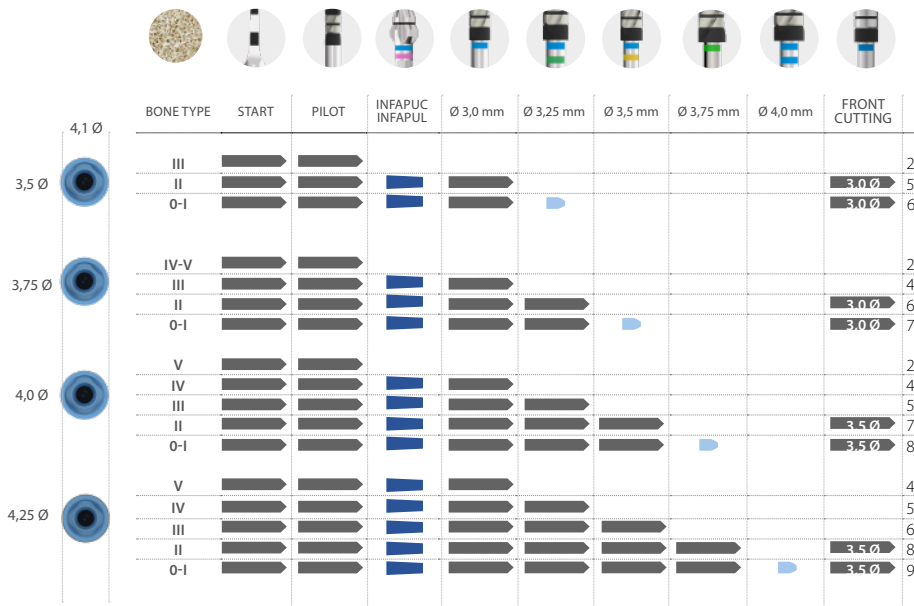


Standard length implants

(from 8,5 to 15 mm)




 Total length of the implant
 6,5 mm drilling depth
 3 mm drilling depth

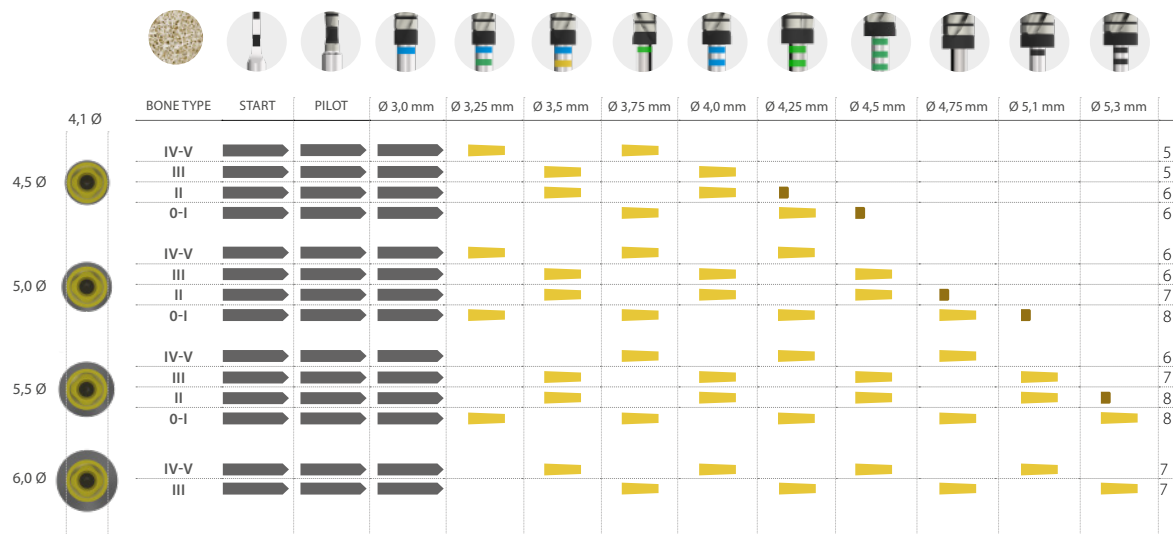

Short implants

(4,5 - 5,5 - 6,5 - 7,5 mm)

 Total length of the implant
 Total length of the implant with specified diameter drill
 3 mm drilling depth

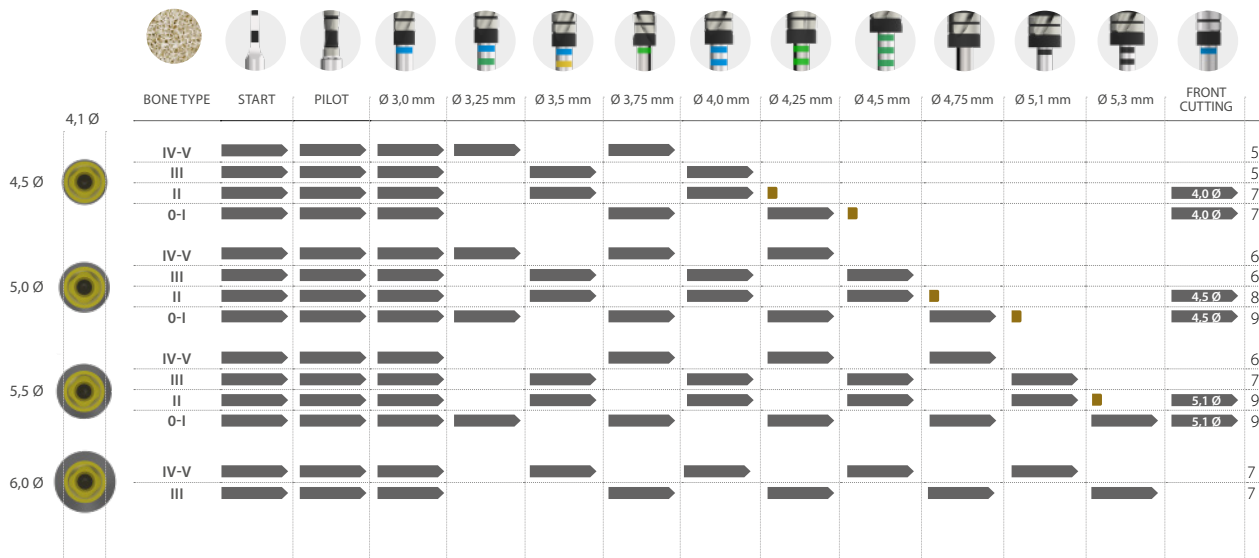




Standard length implants

(from 8,5 to 15 mm)



- Total length of the implant
- 75 % of the total length of the implant
- 25 % of the total length of the implant
- 5,5 mm drilling depth
- 4,5 mm drilling depth

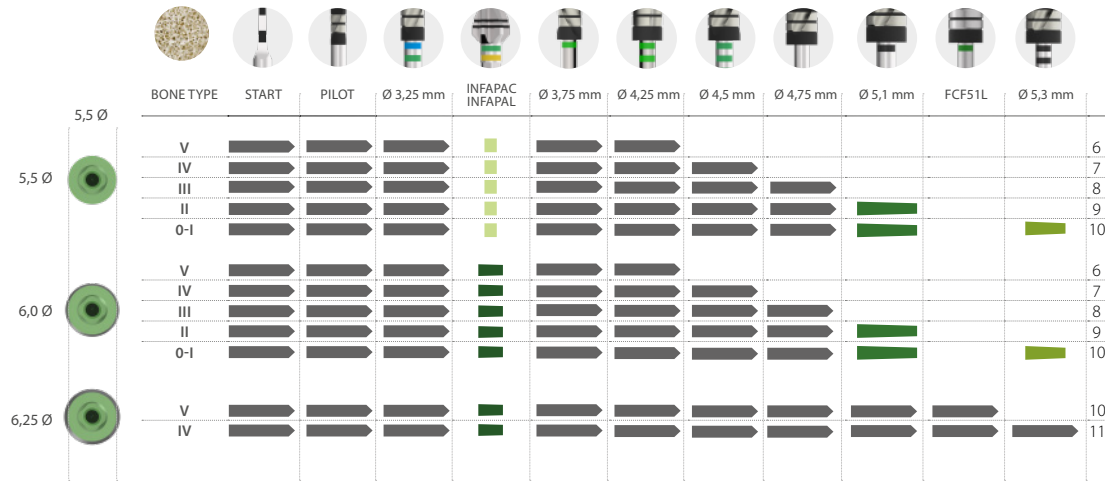

Wide Platform
Platform diameter Ø5,5 mm

drilling
sequence guide






Short implants

(5,5 - 6,5 - 7,5 mm)

- Total length of the implant
- X,X Ø Total length of the implant with specified diameter drill
- 5,5 mm drilling depth
- 4,5 mm drilling depth
- 2-3 mm drilling depth

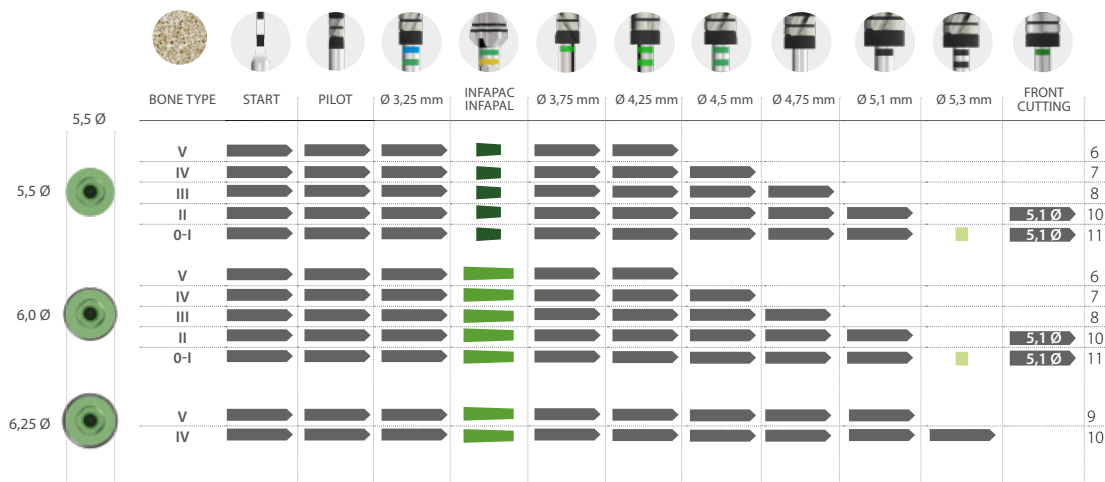

Wide Platform
Platform diameter Ø5,5 mm

drilling
sequence guide

bti
Human
Technology